



SURE & EARLY
No more guesswork from confusing lines


SureEarly Digital Multi-Use Pregnancy Test shows you **easy-to-read & clear results**

Korea's the First & No. 1 Digital Pregnancy Test

Surprisingly clear results in under 3 minutes
Clearer precision than ever before
Over 99% accurate results appear in 3 minutes

What's special with SureEarly™ Digital Multi-Use Pregnancy Test?

- The digital results screen (LCD) is visible, taking away the confusion of interpreting lines.
- More chances of detecting your pregnancy over 99% accurate without misreading.
- Shows user's procedural errors on the LCD display during testing for the best and most accurate result.
- Multiple use test type saves your cost.

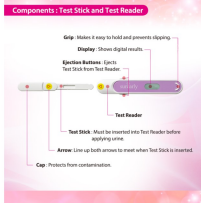

When to Test

- For the best and most accurate results, we recommend that you test on the day your period is due.
- You can test your urine anytime of the day but, generally a first morning urine sample has the highest concentration of hCG.
- Excessive fluid intake needs to be avoided before testing.

How to Use

Step 1. Assemble the test

1 Take off the cap. The test stick must be used immediately after opening the test pouch.

2 Insert the Stick.

Assemble the test by inserting the stick into the test reader. Make sure correct alignment on the Reader must be for use.

Do not use until the Water Drop control appears. Do not use until the Water Drop control appears. Do not use until the Water Drop control appears. Do not use until the Water Drop control appears. Do not use until the Water Drop control appears.

Step 2. Test in your urine

Either: Immerse the Absorbent Tip directly into urine. Hold for 10 seconds.

OR: Apply the urine stream directly into the Absorbent Tip for 3 seconds.

Be careful not to make the Test Reader too wet.

Step 3. Wait

Replace the cap and hold the Test Reader on a flat surface with the Display face up.

The Water Drop symbol will start blinking to show the test is working. Avoid the Absorbent Tip touching the urine in the test stick.

Do not hold the Test Reader with the Absorbent Tip pointing upwards. Be careful not to press the Ejection Buttons while the test is working.

Step 4. Get Your Results in about 3 Minutes

YES means your urine contains the pregnancy hormone hCG. Contact your physician to confirm that you are pregnant.

NO means your urine does not contain the pregnancy hormone hCG. Pregnancy may not be present. Please check the instructions for further information.

hCG levels will remain in the Urine for 72 hours. Test once again to confirm the result for 72 hours.

Device Management after use

Disconnecting the Test Stick: When you press the Ejection Buttons at the top, the Test Stick is ejected automatically from the Test Reader. Please do not touch the Test Stick. Please do not touch the Test Stick. Please do not touch the Test Stick. Please do not touch the Test Stick.

Using the Test Reader again: When the Test Stick is ejected with urine, please wipe it clean with dry tissue. When the Test Stick is ejected, you can reuse the Test Reader.

Cleaning the Test Reader: Clean the Test Reader after using the Test Stick in the pack. Washed the battery from the Test Reader and open the battery pack and the Test Reader according to the local regulations.

Q&A

- When can I test?** You can test 3 days before your expected period for pregnancy test. But the best time is the day of your expected period. hCG increases as you get closer to the menstrual cycle. You can test any time of the day after the due date.
- Can I have sex a few days before my period?** You can have sex a few days before your period. It is generally that morning urine contains the highest concentration of hCG.
- How accurate is the test?** SureEarly™ Digital Multi-Use Pregnancy Test is more than 99% accurate in laboratory tests. When used correctly, the test is 99% accurate in real life.
- Can I take medicine that may affect the test results?** No. The test is designed to be used by anyone. No matter what you are taking, you can use the test. It is not intended for visual observation. You must use the digital result (YES/NO) in the display.
- Can my test be "NO" though I am not pregnant?** False pregnancy, also known as pseudopregnancy, is a condition where a woman's body produces hCG levels similar to those of a pregnant woman. This can lead to a false positive result on a pregnancy test. However, this is a rare condition and usually resolves itself within a few weeks. If you are not pregnant, you should not use the test. It is not intended for visual observation. You must use the digital result (YES/NO) in the display.
- Can I use the Test Reader with Test Sticks from other packages?** This device is only for use with the Test Sticks supplied with the Test Reader. It is not intended for use with Test Sticks from other packages.
- My test has said "YES" from the first test, but it said "NO" from the next test after a few days. What does it mean?** It is possible that a natural loss of the pregnancy which can occur in the early stage.