

surearly
Digital Pregnancy Test

FDA CE ISO13485

100% Risk-free - Europe CE certified
 MFDS (Korea) approved - ISO 13485 certified
 KGMP certified

www.surearly.com

SURE & EARLY
 No more guesswork from confusing lines

Surearly Digital Pregnancy Test shows you easy-to-read & clear result

Korea's the First & No. 1 Digital Pregnancy Test

Intelligently show results in words and numbers
 User's procedural error detection
 Over 99% accurate results against 112,000 cases

What's special with Surearly™ Digital Pregnancy Test?

- The digital display shows YES/NO for results, taking away the confusion of interpreting lines.
- More chances of detecting your pregnancy over 99% accurate without compromising.
- Show user's procedural errors on the LCD display during testing for the best and more accurate result.



When to Test

- For the best and most accurate results, we recommend that you test on the day your period is due.
- You can test your urine anytime of the day but, generally the first morning urine contains the highest concentration of hCG.
- Excessive fluid intake needs to be avoided before testing.

How to Use

Step 1: Preparation before starting the test

- 1 Take off the cap.**

 -Use it immediately after removing the foil pouch.
- 2 Remove the Power Film.**

 -Water Drop symbol on the display means the test is ready.
 -Do not use until the Water Drop symbol appears.
 -When Water Drop symbol appears, apply urine right away.
- 3 Ready to test.**

Step 2: Test in your urine

Either immerse the Absorbent Tip directly into the urine in a container for **10 seconds**.

OR apply the urine stream directly into the Absorbent Tip for **5 seconds**.

-Be careful not to make the Test Reader too wet.

Step 3: Wait

Place the test end by the Test Reader on a flat surface with the Display facing up.

- The Water Drop symbol will start blinking to show the test is working.
- Keep the Absorbent Tip pointing downwards on the test stick.
- Do not hold the Test Reader with the Absorbent Tip pointing upwards.
- Do not touch or press the Detection Buttons while the test is working.

Step 4: Get Your Results in about 3 Minutes

3 minutes

Pregnant **Not Pregnant**

YES means your urine contains the pregnancy hormone hCG. Consult your physician to confirm that you are pregnant.

No means your urine does not contain the hCG and you may not be pregnant. Please wait for the instructions for care in the instructions.

-YES/NO / Test Error symbol will remain on the Display for 48 hours.

Q&A

- When can I test?**
 The best time is the first day of the expected period. You can test anytime of the day after the due date.
- How accurate is the Test?**
 Surearly Digital Pregnancy Test is more than 99% accurate in laboratory tests, when used from the day you expect your period to start.
- Do I have to use first morning urine?**
 You can test with your urine anytime of the day. However, generally the first urine of the day contains the highest concentration of hCG.
- Do any drugs or alcohol affect the test?**
 Over-the-counter drugs containing hCG hormone can affect the result. However, it does not affect the result after 7-14 days of the ovulation drug treatment is completed.
 The test is not affected by hormones which do not contain hCG. The test is also not affected by oral contraceptive pills, antibiotics, vitamin supplements, birth control pills and hormone replacement therapy.
- Can we test for "Yes" though we are not pregnant?**
 A number of conditions other than pregnancy can cause elevated levels of hCG and may give a false "YES" result. If you receive unexpected or inconsistent results, please consult with a physician.
- My test has said "YES" even the Test End, but it said "NO" from the next test after a few days later. What is the reason?**
 It is possible for a false result from the pregnancy which can occur in the early stages.